Note de service

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Anti-Wandering Solutions
SMART
Quieter Fall Prevention™

**To Ensure Resident Safety,
Replace This Pad When Pad Life Expires**

The Smart Change Pad Indicator™ System (patented technology) automatically alerts caregivers when the sensor pad's life has expired. Change Pad Indicator™ technology utilizes an internal clock which counts down the life of the sensor pad while the pad is plugged in and pressure is applied to it.

When the Sensor Pad has expired, the Change Pad Indicator™ will emit a continuous chirping sound and the fall monitor will illuminate. Replace the sensor pad.

NOTE: This Change Pad Indicator™ Sensor Pad is compatible with all Smart fall monitors but will only indicate when a Timed Sensor Pad™ has expired when used with a Change Pad Indicator™ Monitor.

CAUTION: The manufacturer does not recommend the use of this device to stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program.

TB-RI	TB-90R	TB-45R
1 Year Life Pad	90 Day Life Pad	45 Day Life Pad

100% Latex Free Product

Start Date: 21/10/21 End Date: 21/10/22

Use start date to track pad use.
*Pad must be replaced when pad life expires

**Change Pad Indicator
TIMED Bed Sensor**

Pad Use Instructions

1. Remove the protective boot. Install new batteries* into the Smart fall monitor. The battery compartment door is secured with a Phillips-head screw. Please refer to monitor instructions for battery requirements.
2. Plug the cord end of the sensor pad into the bottom of the Smart fall monitor. Pinch the cord together and loop it through the strain relief recess on the fall monitor (near plug input).

Care & Maintenance

- To clean pads, use disinfectant.
- When using a sensor pad, use memory foam, test properly.
- To minimize the spread of infection, each patient is recorded.
- An incontinence pad must be tested prior to use.
- **DO NOT** immerse the pad in water.
- **DO NOT** expose the pad to heat.
- **DO NOT** fold.

Troubleshooting

1. Make sure that the sensor pad is properly connected.
2. Check the power supply. Replace new batteries. Or, if you are using an adapter, plug the adapter into the power outlet.
3. Check the side of the sensor pad. A. If switch is in "Off" position, press Reset button to "On". B. If the switch is in "On" position, use a Caregiver Key to reset the sensor pad. C. If there is an "On" position, the sensor pad will not reset the sensor pad.

CE
Tested and approved to IEC 60601-1-2 Medical Devices General

Sensor Pad positioned under shoulders or under hips and connected to fall monitor.

Durée de vie

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